Instructions for Use:

Test to be conducted weekly, preferably daily, in an EMPTY tank.

1. Ensure that the Ultrasonic Test Holders are clean and dry.
2. Ensure user hands are clean and dry.
3. Place one VERIFY™ Ultrasonic Indicator in each of the five recommended holders. The indicator should be placed in the spring in the inside of the holder.

4. We recommend attaching one holder to each of the four corners of the tray using the clip on the back of the holder and recommend setting one holder in the center of the tray.

Example of recommended arrangement in tray shown below:

Ultrasonic Tray

5. Place tray into the ultrasonic unit and run a cycle.
6. After the cycle is completed, remove the holders from the tray and carefully remove the VERIFY™ Ultrasonic Indicator from each holder. Caution should be used, as the device may be hot and any residual soil may stain.

7. Inspect the indicator for evidence of soil by placing the plastic film against a white background.

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Before Use / Avant utilisation

Pass / Conforme

If all soil has been removed from the indicators, continue using unit as normal.

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Fail / Non conforme

Fail / Non conforme

If evidence of soil remains on the indicator, departmental procedures should be followed in respect to cleaning efficacy standards.

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